

IFU ADDENDUM FOR Z845 R13 and R14

This update outlines the error identified in the Instructions for Use (IFU) for PENTAX Medical ultrasound gastroscope models EG-3670URK and EG-3870UTK. Please include this addendum in all copies of Z845 R13 and Z845 R14, and have users read and carefully follow this addendum when using these endoscopes models.

Page	Error	Correct
p70	WARNING: The <u>cleaning detergent solution should</u> remain in contact with the ALL internal channels and external endoscope surfaces for the time period recommended by the manufacturer of the disinfectant.	WARNING: The <u>disinfecting solution must</u> remain in contact with the ALL internal channels and external endoscope surfaces for the time period recommended by the manufacturer of the disinfectant.